

Beautiful[®] II PINK & WHITE

FLUORIDE RELEASING, BIOACTIVE,
NANO-HYBRID COMPOSITE

VISIT WWW.SHOFU.COM

SHOFU

OR CALL 800.827.4638



Photos courtesy of Frank J. Milnar, DDS, AAACD



Photos courtesy of Sushil Koirala, BDS, AAAD

gingiva & enamel shades

Giomer PRC
Technology


miCD
Minimally Invasive
Cosmetic Dentistry

Beautiful® II PINK & WHITE


The award-winning nano-hybrid composite, **Beautiful II**, delivers restorations that resemble natural teeth, are reversible and cost-effectively repairable. This fluoride releasing and recharging bioactive composite demonstrates enhanced thixotropic properties for excellent marginal adaptation and efficient contouring without slumping or sticking. The material's high compressive and flexural strength of ca. 330 MPa and 130 MPa, respectively, whether placed in the anterior or posterior regions, is ideal for preserving long-term functional stability in the oral environment.

Developed as a functional extension to the existing **Beautiful II** line, the gingiva and enamel colored composites provide general practitioners with the ability to offer highly aesthetic and long-lasting restorations more efficiently and at a lower cost. The new shade modules are highly filled, demonstrate excellent flexural and compressive strength, maintain optimal color stability before and after curing, and exhibit high luster.

- Sustained fluoride release and recharge
- Natural fluorescence
- High level of radiopacity

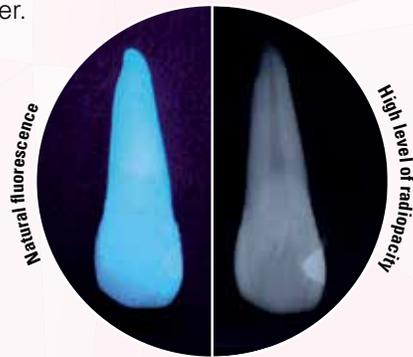

- Superior handling – easy sculpting without slumping or sticking
- Exceptional marginal seal
- Anti-plaque affect minimizes plaque adhesion, inhibits bacterial colonization and plaque accumulation

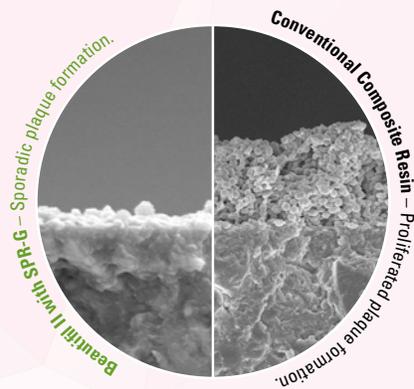

- Fast and easy polishing creates outstanding surface luster that remains over time
- 13 years of clinical success

GIOMER TECHNOLOGY



taking dentistry to the next level

The Beautiful resin composite lines contain Shofu's proprietary Gioner technology, a bioactive surface pre-reacted glass (S-PRG) filler that actively releases six beneficial ions including fluoride.

Gioner materials have been clinically vetted in a series of long-term clinical trials. An eight-year clinical research recorded 100% retention rate with intact esthetics, no secondary caries, no failures and no post-operative sensitivity.¹ A thirteen-year recall study demonstrated 96% of restorations with no secondary caries, in a 66% retention rate.²

Gioner's unique ability to not only release but recharge fluoride helps to provide continual protection to the tooth. Gioner benefits:

- Decrease in acid production of cariogenic bacteria
- Formation of an acid-resistant layer
- An anti-plaque effect
- Reduction in tooth mineral solubility

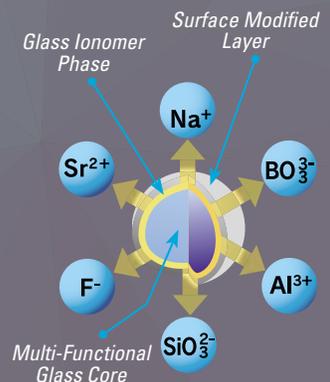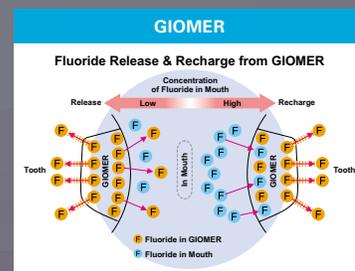

1. Gordan V.V., et al. A clinical evaluation of a self-etching primer and a Gioner restorative material: Results at 8 years. JAM Dent. Assoc. 2007;138:621-627

2. Gordan V.V., et al. A clinical evaluation of a Gioner restorative system: 13-year recall. Presented at IADR General Session; March 23, 2013; Seattle, Washington


PINK POSSIBILITIES

Beautiful II Gingiva, provides general practitioners with the means to optically improve gingival aesthetics in the operatory, thus simplifying and reducing the time and cost of a treatment.

Five gingival shades of **Beautiful II** can be easily blended to custom shade-match any type of gum tissue to meet the needs of all ethnicities. The material can be predictably bonded to dentin, enamel, ceramics and metal substrates.

Along with aesthetic gingival reproduction to the cervical area, **Beautiful II Gingiva** can be utilized by clinicians to camouflage the exposed implant abutments and crown and bridge margins, to repair and re-contour the gingival portion of indirect restorations, to modify the aesthetics of provisional restorations and to improve soft-tissue appearance in denture patients by revitalizing a worn or fractured denture.

- Restore recession, lost papilla, black triangles
- Mask exposed abutments, C&B margins
- Repair worn, fractured dentures
- Custom shade-match, finish/polish for natural results
- Aesthetic solutions without surgery
- Perfect for Class V restorations

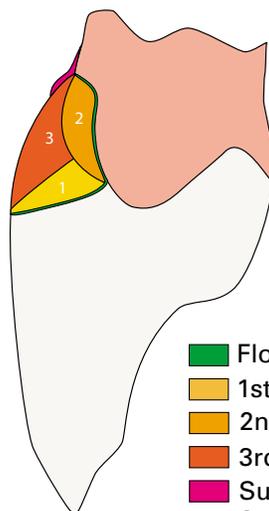

Before

Photos courtesy of Sushil Koirala, BDS, AAAD



After. Periodontal graft & restoration with BEAUTIFIL II Gingiva



Transmi



WHITE ESTHETICS



Photos courtesy of Sushil Koirala, BDS, AAAD



Photos courtesy of Frank J. Milnar, DDS, AAACD

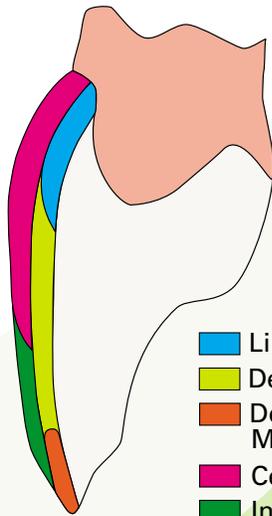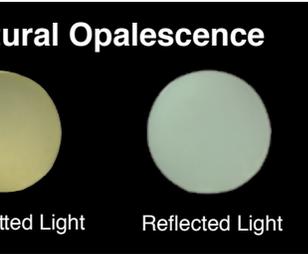


- Lingual Enamel
- Dentin Layer
- Dentin Layer with Mamelons & Incisal Halo
- Cervical Enamel
- Incisal Enamel

Intended for direct anterior restorations, **Beautiful II Enamel**, is available in four shades that can be layered and blended together to produce highly aesthetic results.

The optical characteristics of the enamel tones with chameleon-effect reproduction allow for creating indistinguishable restorations with aesthetics of natural teeth, providing the clinicians a cost-effective alternative to ceramic veneers.

- *Achieve highly aesthetic direct restorations with minimal shade selections*
- *Create natural results with a simplified layering and sculpting method for shade modification*
- *Modeled after the porcelain color concept—easy color matching*
- *Direct veneers made easy*

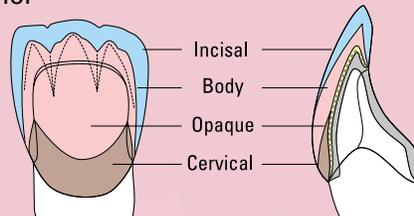

layer & sculpt technique



Enamel shell created with BEAUTIFIL II Enamel HVT

Mamelon structure created with BEAUTIFIL II A3O

Dentin built up with BEAUTIFIL II A2 (middle-third) and A2 BEAUTIFIL II Enamel at incisal edge



Outer Enamel layer created with BEAUTIFIL II Enamel at incisal edge

Photos courtesy of Sushil Koirala, BDS, AAAD


SEVEN SHADES OF ENAMEL

PHYSICAL PROPERTIES


- Compressive Strength: 354 Mpa
- Flexural Strength: 130 Mpa
- Vickers Hardness: 62 Hv
- Wear Resistance: 0.52 wt%
- Filler Load: 83.3 wt%
- Radiopacity: 3.4 Al : mm
- Fluoride Release & Recharge: Yes

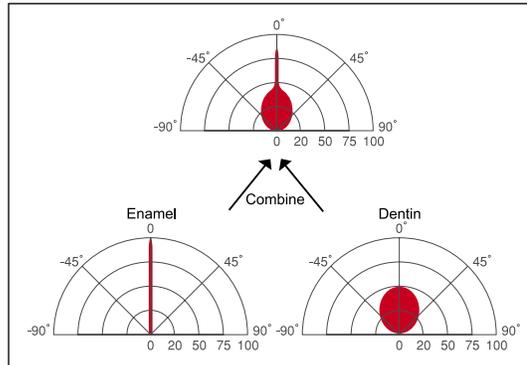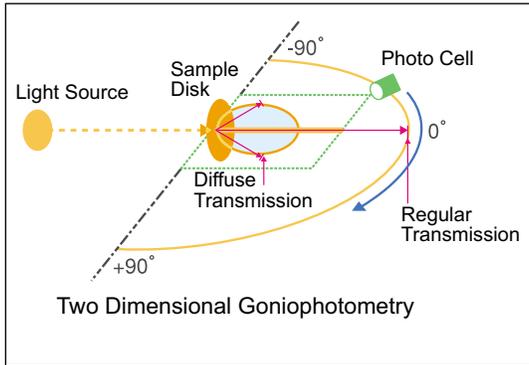

Surface Gloss (GV)

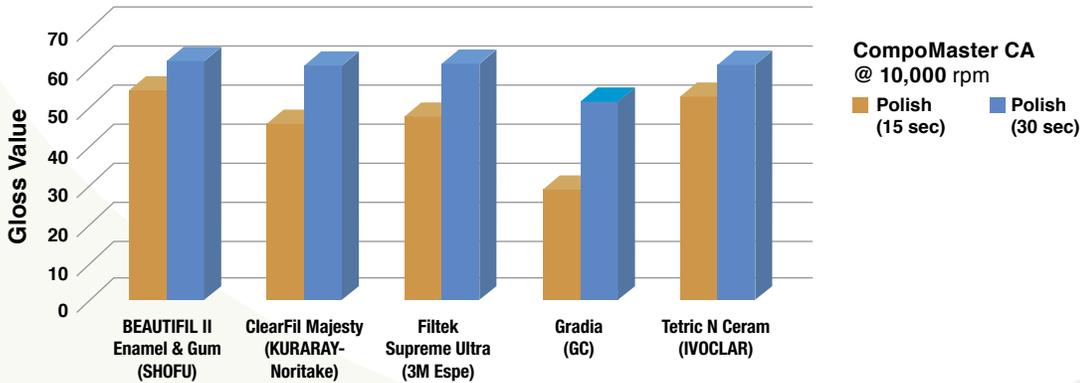

Translucency (Pre & Post-cure)
Shift in Translucency

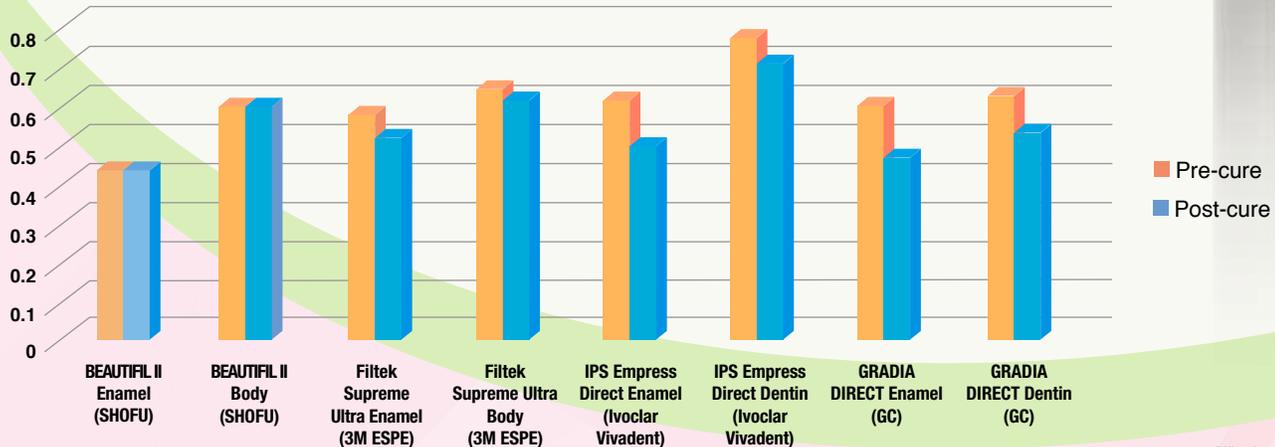

*All trademarks belong to their manufacturers



Beautiful® II PINK & WHITE

VISIT WWW.SHOFU.COM

SHOFU

OR CALL 800.827.4638



	G-Dark Pink	G-Light Pink	G-Brown	G-Orange	G-Violet
2.5g					
PN	Y2257	Y2256	Y2259	Y2260	Y2258

	High-Value Translucent (HVT)	Translucent (T)	Low-Value Translucent (LVT)	Amber (AM)
2.5g				
PN	Y2251	Y2250	Y2252	Y2253

Beautiful II PINK Kit

Part Number **GMK1**

- 5 (2.5g) Beautiful II Gingiva syringes
- 1 (2g) Beautiful Flow Gingiva F02 syringe

Bonus Items Included!

- 4 Beautiful Flow Plus F00 & F03 A2 Tips
- 2 Beautiful II A2 Tips

COMPLEMENTARY PRODUCT

PINK Prep Kit

Part Number **8205K1**

- 2 Minimally Invasive FG Robot Diamonds
- 3 FG Robot Diamonds
- 5-Hole Ruby Pink BurButler


Beautiful II PINK & WHITE Kit

Part Number **GMK3**

- 5 (2.5g) Beautiful II Gingiva syringes
- 1 (2g) Beautiful Flow Gingiva F02 syringe
- 4 (2.5g) Beautiful II Enamel syringes

Bonus Items Included!

- 10 Beautiful Flow Plus F00 & F03 A2 Tips
- 10 BeautiBond Unite Dose
- 10 Super-Snap Singles Mini

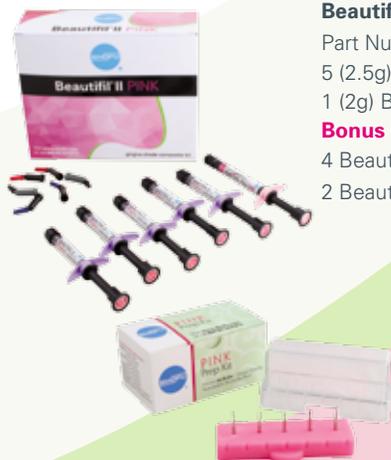

USA

Shofu Dental Corporation
1225 Stone Drive
San Marcos, CA 92078-4059, USA

JAPAN

Shofu Inc.
11 Kamitakamatsu-cho
Fukuine, Higashiyama-ku
Kyoto 605-0983, Japan

SINGAPORE

Shofu In. Singapore Branch
10 Science Park Road #03-12
The Alpha, Science Park II
Singapore 117684

U.K.

SHOFU UK Riverside
House,
River Lawn Road,
Tonbridge,
Kent, TN9 1EP, UK

CHINA

Shofu Dental Supplies
(Shanghai) Co. Ltd.
No.645 Jiye Road
Seshan Industry Park
Songjiang 201602,
Shanghai, China

GERMANY

EU Representative:
Shofu Dental GmbH
Am Brüll 17
40878 Ratingen, Germany