



# FAST & CONSISTENT UNIVERSAL BONDING

**G-Premio BOND™**  
Universal, 8th Generation Bonding Agent


- New bottle design with flip-top cap for better dispensing control
- Flexible for use in total, selective and self-etch bonding techniques
- Fast application (10 seconds)
- Excellent stability with extended working time (up to 5 minutes)





## Many Changes To Create A Better Bonding Agent

The dispersion of silica filler was improved and viscosity was reduced to increase working time by 333%\*. The amount of water was increased to dissolve and remove smear layer better and the amount of carboxylic monomer was increased to keep acidic pH. The amount of photoinitiator was also increased to make a firm bonding layer. These adjustments all result in a faster application and shorter treatment time. The new bottle contains a unique, silicone cover. This redesign offers many benefits such as a more precise drop by drop dispensing to eliminate waste and keeps heat from fingers from impacting bonding performance by serving to insulate the bonding agent.

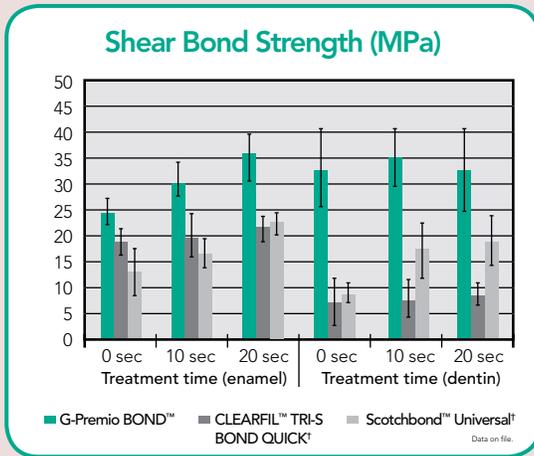

Apply and wait 10 sec.

Dry for 5 sec. (MAX air pressure).

Light-cure for 10 sec.

Place the composite resin.

	Distance From Light Guide Tip	
	< 10 mm	> 10 mm
Halogen/LED (700 mW/cm <sup>2</sup> )	10 sec	20 sec
High power LED (more than 1200 mW/cm <sup>2</sup> )	5 sec	10 sec

Universal Bond		
Indication		Features
Direct Restoration		Reliable bond to enamel and dentin
		All etching modes (self, selective, and total etch)
		Intraoral repair (only metal and Zr)
		Shorter application time (0 wait time)
Indirect Restoration		Bonding to enamel and dentin
	+ Dual Cure Activator	Thin film thickness (3µm) allows light cure technique
	+ Multi primer	Can be made dual cure by mixing with Dual Cure Activator
		Reliable primer for all substrates



Self-etch courtesy of Dr. Ibata, Japan



Selective-etch courtesy of Dr. Lassocinski, Poland



Total-etch courtesy of Dr. Browet, Belgium



### 012691 G-Premio BOND™ Kit

Contains: One bottle (5mL), 50 disposable applicators and 20 disposable dispensing dishes.

### 012696 G-Premio BOND™ Bottle Refill

Contains: One bottle (5mL).

### 009276 G-Premio BOND™ Unit Dose Refill

Contains: 50 unit doses (0.1mL) and 50 disposable applicators (fine).

### 009552 G-Premio BOND™ Dual Cure Activator (DCA) Refill

Contains: One bottle (3mL).